






PAEDIATRIC VA TEST CARDS

These *flash cards* are used for assessing Visual Acuity (VA) of young children who cannot reliably read letters or numbers. The symbols (called *optotypes*) have been tested on young pre-school children for recognition.

All cards are suitable to be printed, cut to size and used in a clinical setting. Most practices prefer to laminate them for durability, while others simply re-print and replace them as needed.

TABLE OF CONTENTS

	DEVELOPMENT OF THE HANKS PAEDIATRIC TEST CARDS.....	3
	FLASH CARDS INTRODUCTION.....	6
	How To Use These Pages.....	7
	Recommended Selection of Test Cards.....	8
	Clinical Equipment	9
	Anaglyph Test Sheet.....	10
	THE HANKS PAEDIATRIC TEST CARDS	11
	Paediatric Optotypes (Symbols).....	12
	Paediatric Symbols with Contour Interaction Bars.....	48
	Illiterate E.....	80
	Landolt C.....	116
	OTHER TEST CARDS	134
	Paediatric Duochromes.....	134
	Worth 4 Dot	136
	Traffic Lights	137
	Fixation Disparity	138
	FLASH CARDS APPENDIX	139
	Converting the Measured VA for the Testing Distance	140
	Snellen Visual Acuity Conversion for Notation	141
	Eyecare Education Series - Using Paediatric Test Cards	142
	Eyecare Education Series - Why the Contour Interaction Bars Are Red	145
	Fast-Find Index.....	146
	References	147
	Our Digital Resources for Optometrists	148